




# ICV

## INSTRUCTIONS FOR USE




### Fill the ICV Tank


Fill the tank when the low fluid indicator is lighted.

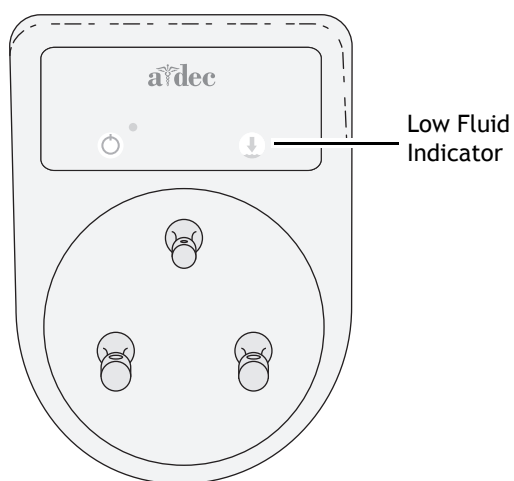

1. Pull the blue water tubing and the water level connector off of the tank's connectors.

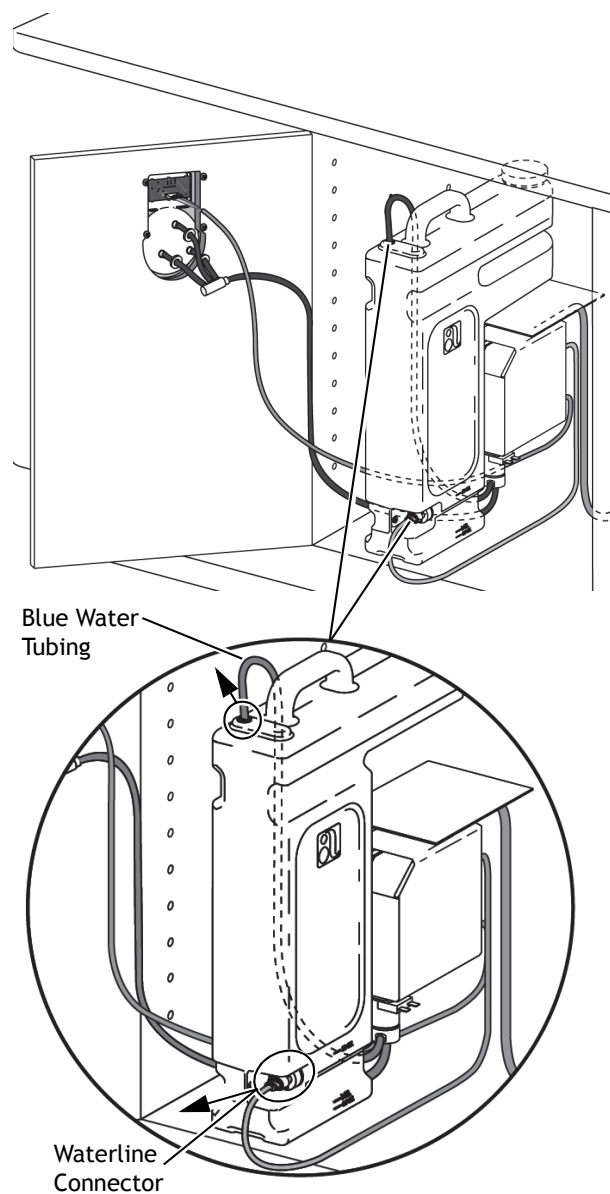

2. Remove the cap from the tank and add cleaner and water per the cleaner manufacturer's recommendation.

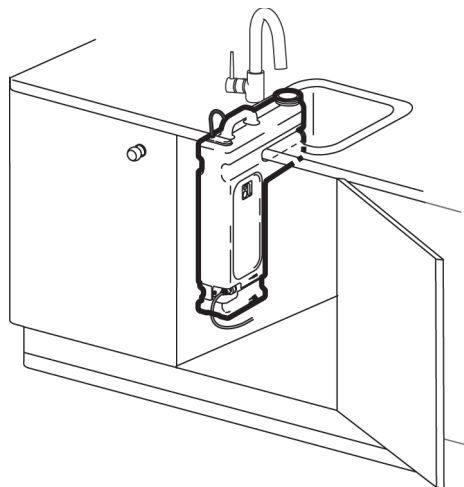

3. Reinstall the cap on the tank.
4. Reconnect the water tubing and the water level connector on the tank's connectors.

## Clean the Vacuum Instrumentation




**CAUTION** Before you press the start/stop button, ensure that at least one tubing is attached and the tubing valve is open. This prevents damage to the cabinet face.

1. Connect the handpiece tubing to the ICV.
2. Open the valve(s).
3. Press the start/stop button.



**NOTE** This product is rated 100-120/220-240 VAC, 50-60 Hz, 0.5 A.

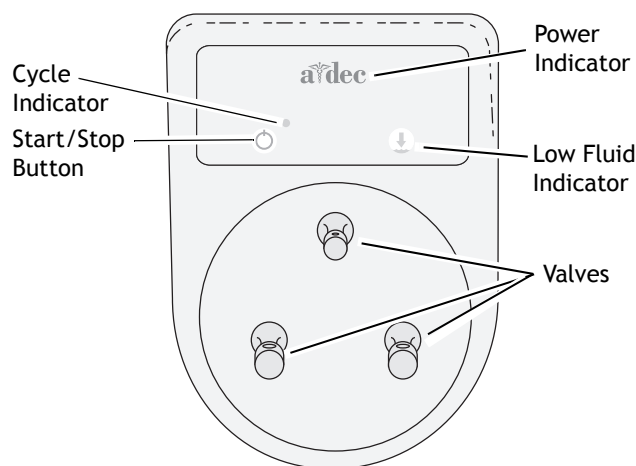

### A-dec Headquarters

2601 Crestview Drive  
Newberg, OR 97132 USA  
Tel: 1.800.547.1883 Within USA/CAN  
Tel: 1.503.538.7478 Outside USA/CAN  
Fax: 1.503.538.0276  
[www.a-dec.com](http://www.a-dec.com)

### A-dec Australia

Unit 8  
5-9 Ricketty Street  
Mascot, NSW 2020  
Australia  
Tel: 1.800.225.010 within AUS  
Tel: +61.(0).2.8332.4000 outside AUS

### A-dec China

A-dec (Hangzhou) Dental Equipment Co., Ltd.  
528 Shunfeng Road  
Qianjiang Economic Development Zone  
Hangzhou 311106  
Zhejiang, China  
Tel: +1.503.538.7478

### A-dec United Kingdom

Austin House, 11 Liberty Way  
Nuneaton, Warwickshire CV11 6RZ  
England  
Tel: 0800 ADECUK (233285) Within UK  
Tel: +44 (0) 24 7635 0901 Outside UK

## Regulatory Information

Regulatory information mandated by agency requirements is provided in the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00), which is available in the Document Library at [www.a-dec.com](http://www.a-dec.com).



86.0617.00 Rev F  
Copyright 2016 A-dec Inc.  
All rights reserved.  
IFUev6